

SwabCap[®] has you covered!



Between line accesses, use SwabCap[®] to protect and disinfect your luer access valves.



Luer Access Valve Cap
with 70% IPA as a
disinfectant.

Single use only
Latex Free
DEHP Free
Preservative Free

SwabCap[®]


- Acts as a physical barrier against touch and airborne contamination
- Passively disinfects the luer access valve between line access
- Is sterile, individually packaged
- **Single use / DO NOT REUSE**
- **Remains in place between treatments**

See reverse for instructions in case removal is needed for medication administration.



SwabCap®

To Remove for access to valve

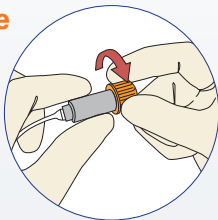

Two-handed removal must be used to make sure luer access valve is not removed unintentionally.


Grasp luer access valve and twist counter clockwise away from valve. Remove and discard after each use.

Inspect luer access valve to verify that it's connected securely to the catheter.

Valve is ready for access.


No further swabbing is necessary after removal.

Peel.

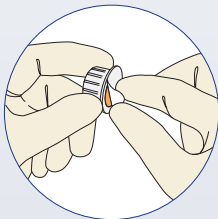


Remove protective cover by pulling tab. **Do not remove SwabCap® from white plastic holder.**

Twist.

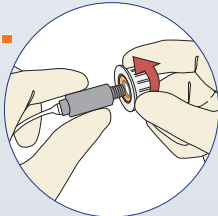

Hold luer access valve in one hand while pushing and twisting the SwabCap® in a clockwise direction onto the luer access valve.

Protect.

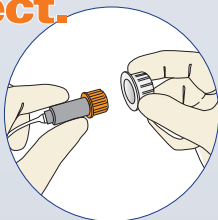

After SwabCap® has been twisted into place, gently pull white plastic holder off luer access valve.

Manufactured by



www.excelsiormedical.com