

SwabFlush™


Your final saline flush is now your first line of defense against infection.

Use aseptically – For IV flush only.

Follow Excelsior Medical's instructions for use and your institution's procedures for IV flush administration.


SwabCap® and saline flush syringe are both for SINGLE USE ONLY

SwabCap® is for use on swab-able needleless connectors, as a protecting cap that provides a physical barrier to contamination between line accesses and as a disinfecting cleaner prior to line access.

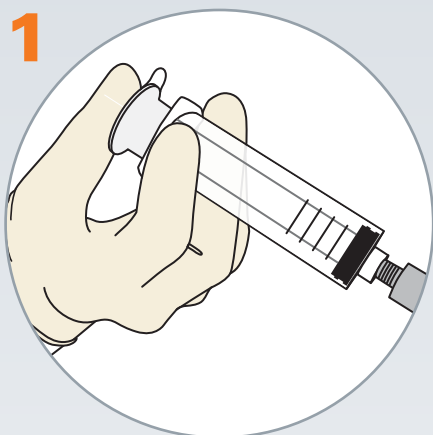


1 Flush site with an appropriate volume of solution (determined by your institution) and according to the manufacturer's instructions for use, for maintaining catheter patency.


Ensure that all the fluid has been flushed or discarded to avoid squirting when applying the SwabCap.

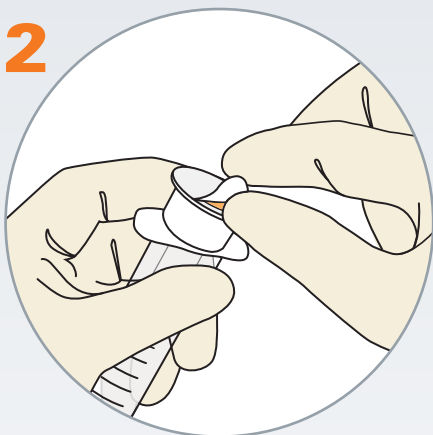


2 Hold the back of the plunger firmly and remove the protective cover by pulling the tab. Do not remove SwabCap from white plastic holder.

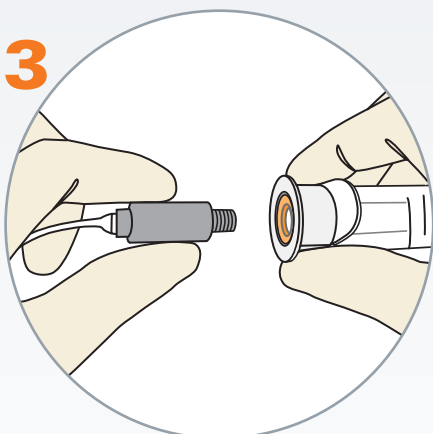


3 Turn syringe so that the back of the plunger is now facing the needleless connector.

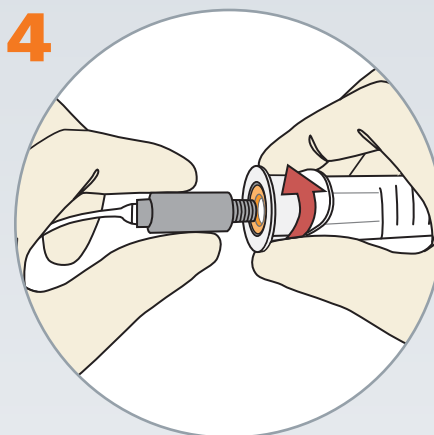

4 Hold needleless connector in one hand while pushing and twisting the SwabCap in a clockwise direction onto the needleless connector.

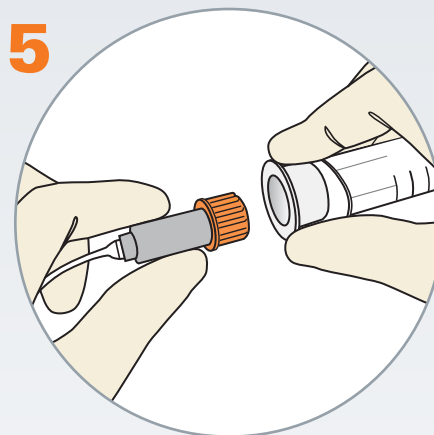

5 After SwabCap has been twisted into place, gently pull syringe off the needleless connector.

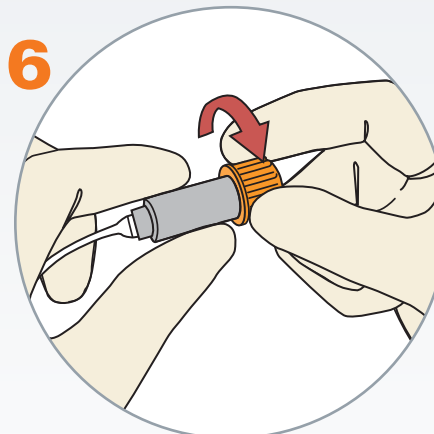

6 Two-handed removal must be used to make sure luer access valve is not removed unintentionally.

Grasp luer access valve and twist SwabCap counter clockwise away from valve. Remove and discard after each use.

Inspect luer access valve to verify that it's connected securely to the catheter.

Valve is ready for access.

Upon SwabCap removal, no additional disinfection is necessary for valve access.

Use in accordance with any warnings or precautions appropriate to the medication being administered. Do not use if compatibility is in doubt.

For Single Use Only - Discard syringe and any unused portion of the solution.



Corporate Headquarters
Excelsior Medical Corporation
1933 Heck Avenue, Neptune, NJ 07753

www.excelsiormedical.com

Toll Free 800-487-4276
Main Phone 732-776-7525
Fax 732-776-7600

E-mail: info@excelsiormedical.com